


Emergency Disconnect Procedure

Model AK 96
For Paramedics
Catheter

1

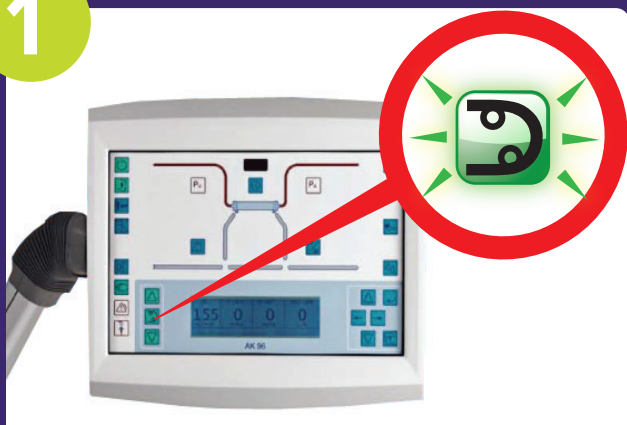


PRESS BLOOD PUMP BUTTON.
Blood pump will stop and
button will start flashing.

2




**USING THE PRODUCTS
FOUND IN THE PATIENT'S
EMERGENCY KIT...**

3

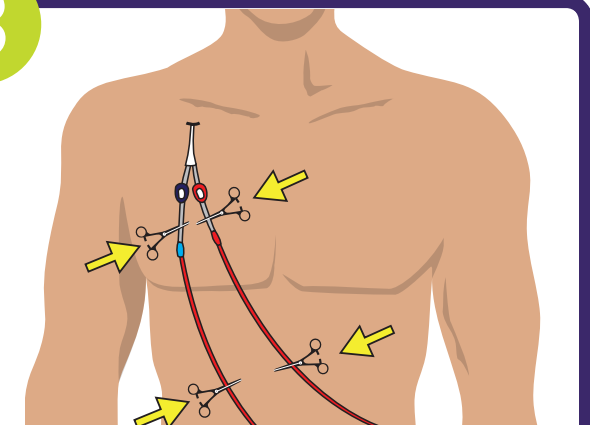

**CLOSE CLAMPS ON THE
CATHETER AND BLOODLINES.
ADD SAFETY CLAMPS BELOW
AND ABOVE THE CONNECTION.**

4

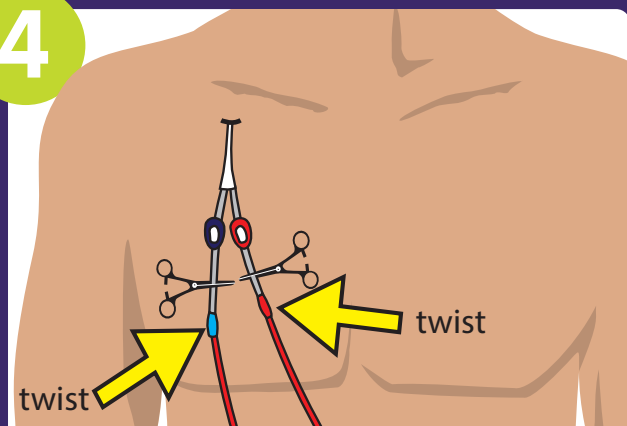

**DISCONNECT LINES AT THE
CONNECTION SITE
BY TWISTING.**

5



**TRANSPORT THE PATIENT
AS-IS TO HOSPITAL.**

Produced by

BCRenal
Provincial Health Services Authority

fraserhealth
Better health. Best in health care.

Interior Health

northern health
the northern way of caring

**Providence
Health Care**

**Vancouver
CoastalHealth**
Promoting wellness. Ensuring care.

island health

**Provincial Health
Services Authority**

**BRITISH
COLUMBIA**