


Emergency Disconnect Procedure

Model AK 96
For Paramedics
Graft or Fistula

1

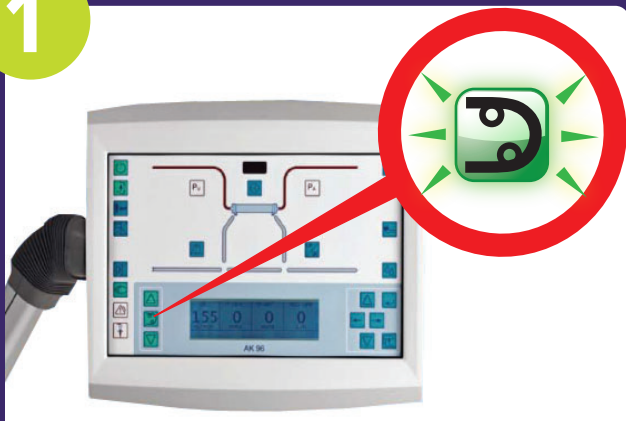


PRESS BLOOD PUMP BUTTON.
Blood pump will stop and
button will start flashing.

2




**USING THE PRODUCTS
FOUND IN THE PATIENT'S
EMERGENCY KIT...**

3

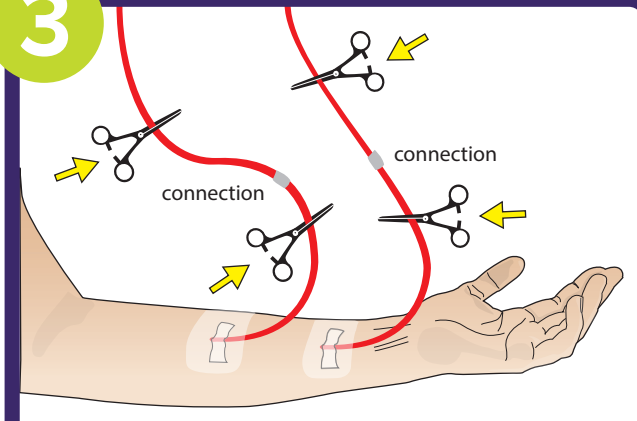

**CLOSE TWO CLAMPS ON NEEDLES
AND BLOODLINES. ADD SAFETY
CLAMPS BELOW AND ABOVE THE
CONNECTION.**

4

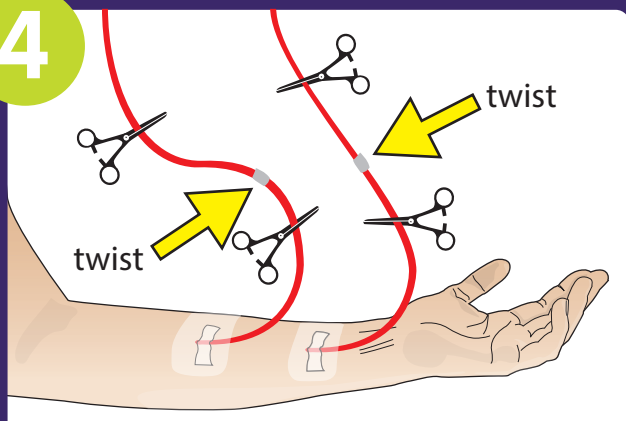

**DISCONNECT LINES AT THE
CONNECTION SITE BY
TWISTING.**

5



**TRANSPORT THE PATIENT
AS-IS TO HOSPITAL.**

Produced by

BCRenal
Provincial Health Services Authority

